

<b>REF</b> 540-2	6x3mL	<b>EXP</b> 2025-02-15	<b>LOT</b> 3384FD6	更新日期： 2022-05 1 / 6
------------------	-------	-----------------------	--------------------	------------------------

批号：3384FD6				
项目\仪器\试剂\方法	单位	均值	+ / - 2 SD	
<b>(17-OHP)17-羟孕酮</b>				
Siemens Immulite1000/2000 Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	ng/mL	1.78	1.41-2.15	
<b>(25-OH-Vit D)25羟基维生素D</b>				
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	nmol/L	77.1	61.7-92.5	
<b>(AFP)甲胎蛋白</b>				
Tosoh AIA 1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	ng/mL	117	95-139	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	134	107-161	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	ng/mL	106	84-128	
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA )	ng/mL	121	97-145	
Caris200 WanTai \ Chemiluminescence Immunoassay (CLIA )	ng/mL	118	89.8-146	
<b>(ALD)醛固酮</b>				
DiaSorin Liaison /XL Dedicated \ Chemiluminescence Immunoassay (CLIA )	pg/mL	167	135-199	
Autolumo A2000/A2000 PLUS Autobio \ Chemiluminescence Immunoassay (CLIA )	pg/mL	278	222-334	
<b>(Anti-Tg)抗甲状腺球蛋白抗体</b>				
Siemens ADVIA Centaur XP Siemens \ Chemiluminescence Immunoassay (CLIA )	IU/mL	64.6	51.8-77.4	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	IU/mL	9.86	7.9-11.8	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/mL	67.6	54.1-81.1	
<b>(Anti-TPO)抗甲状腺过氧化物酶抗体</b>				
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/mL	33.3	26.6-40	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	IU/mL	23.4	18.6-28.2	
<b>(ASD)雄烯二酮</b>				
DiaSorin Liaison /XL Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	3.12	2.43-3.81	
<b>(CA 125)糖类抗原 125</b>				
Caris200 WanTai \ Chemiluminescence Immunoassay (CLIA )	IU/mL	87.8	66.7-109	
Siemens ADVIA Centaur XP Siemens \ Chemiluminescence Immunoassay (CLIA )	IU/mL	64.9	52.3-77.5	
Tosoh AIA 1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	IU/mL	85.7	69.5-102	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/mL	52.7	42.2-63.2	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	IU/mL	61.4	48.8-74	
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA )	IU/mL	62.3	49.9-74.7	
<b>(CA 15-3)糖类抗原 15-3</b>				
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA )	IU/mL	71.2	56-86.4	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	IU/mL	45.3	36.1-54.5	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/mL	52.8	42.2-63.4	
Siemens ADVIA Centaur XP Siemens \ Chemiluminescence Immunoassay (CLIA )	IU/mL	50.6	40.4-60.8	
Tosoh AIA1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	IU/mL	58.1	45.9-70.3	
<b>(CA 19-9)糖类抗原 19-9</b>				
Tosoh AIA 1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	IU/mL	39.4	31.6-47.2	

<b>REF</b> 540-2	6x3mL	<b>EXP</b> 2025-02-15	<b>LOT</b> 3384FD6	更新日期： 2022-05 2 / 6
------------------	-------	-----------------------	--------------------	------------------------

批号：3384FD6				
项目\仪器\试剂\方法	单位	均值	+ / - 2 SD	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/mL	55.7	44.6-66.8	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	IU/mL	46.8	36.6-57	
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA )	IU/mL	319	257-381	
Caris200 WanTai \ Chemiluminescence Immunoassay (CLIA )	IU/mL	80.8	61.4-100	
<b>(CEA)癌胚抗原</b>				
Caris200 WanTai \ Chemiluminescence Immunoassay (CLIA )	ng/mL	11.7	8.89-14.5	
Abbott ARCHITECT i series Abbott \ Enzyme linked immunosorbent assays (ELISA)	ng/mL	11.5	9.3-13.7	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	ng/mL	10.5	8.1-12.9	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	11.1	8.89-13.3	
<b>(CK)肌酸激酶</b>				
Roche cobas 4000/6000/8000 Roche \ Urease-UV-rate	U/L	61.3	49-73.6	
<b>(CK-MB)肌酸激酶-MB (活性)</b>				
Roche cobas 4000/6000/8000 Roche \ Immunoturbidimetry (ITA)	U/L	12	9.6-14.4	
Roche cobas e411 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	12.3	9.84-14.8	
Tosoh AIA 1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	ng/mL	16.8	13.4-20.2	
<b>(Cortisol)皮质醇</b>				
Tosoh AIA1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	ng/mL	225	182-268	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	240	192-288	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	ng/mL	228	178-278	
<b>(C-P)C-肽</b>				
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	ng/mL	2.35	1.89-2.81	
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA )	ng/mL	2.28	1.84-2.72	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	2.31	1.85-2.77	
Tosoh AIA 1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	ng/mL	1.74	1.34-2.14	
Siemens ADVIA Centaur XP Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	1.46	1.18-1.74	
<b>(DGXN)地高辛</b>				
Roche cobas e411 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	2.08	1.66-2.5	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	ng/mL	1.97	0-5.91	
<b>(DHEA-S)硫酸脱氢表雄酮</b>				
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	ng/mL	6330	5000-7650	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	5940	4750-7120	
Tosoh AIA 1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	ng/mL	5710	4530-6880	
<b>(E2)雌二醇</b>				
Tosoh AIA 1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	pmol/L	808	644-972	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	pmol/L	1000	798-1200	
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA )	pmol/L	1040	834-1240	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	pmol/L	968	772-1160	

<b>REF</b> 540-2	6x3mL	<b>EXP</b> 2025-02-15	<b>LOT</b> 3384FD6	更新日期： 2022-05 3 / 6
------------------	-------	-----------------------	--------------------	------------------------

批号：3384FD6				
项目\仪器\试剂方法	单位	均值	+ / - 2 SD	
<b>(EPO)促红细胞生成素</b>				
Siemens Immulite1000/2000 Siemens \ Chemiluminescence Immunoassay (CLIA )	miU/mL	33.1	26.5-39.7	
Beckman AU series Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	miU/mL	45.9	37.3-54.5	
<b>(FA)叶酸</b>				
Tosoh AIA 1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	ng/mL	7.37	5.78-8.96	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	6.94	5.55-8.33	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence enzyme immunoassay (CLEIA)	ng/mL	7.73	6.19-9.27	
<b>(Ferritin)铁蛋白</b>				
Tosoh AIA 1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	µg/L	137	109-165	
Roche cobas 4000/6000/8000 Roche \ Immunoturbidimetry (ITA)	µg/L	190	152-228	
Siemens ADVIA Centaur XP Siemens \ Chemiluminescence enzyme immunoassay (CLEIA)	µg/L	153	121-185	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	µg/L	129	105-153	
DiaSorin Liaison /XL Diasorin \ Chemiluminescence Immunoassay (CLIA )	µg/L	210	160-260	
Caris200 WanTai \ Chemiluminescence Immunoassay (CLIA )	µg/L	182	138-226	
<b>(F-PSA)游离前列腺特异性抗原</b>				
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	5.59	4.47-6.71	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	ng/mL	6.43	5.09-7.77	
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA )	ng/mL	6.92	5.54-8.3	
Caris200 WanTai \ Chemiluminescence Immunoassay (CLIA )	ng/mL	5.74	4.36-7.12	
<b>(FSH)促卵泡刺激素</b>				
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	IU/L	26.8	20.6-33	
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA )	IU/L	34.8	28-41.6	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/L	33.3	26.6-40	
Tosoh AIA1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	IU/L	26.8	21-32.6	
Siemens Immulite1000/2000 Siemens \ Chemiluminescence Immunoassay (CLIA )	IU/L	38.9	31.3-46.5	
Caris200 WanTai \ Chemiluminescence Immunoassay (CLIA )	IU/L	23.6	17.9-29.3	
<b>(FT3)游离T3</b>				
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	pmol/L	13.9	11.1-16.7	
Siemens ADVIA Centaur XP Siemens \ Chemiluminescence Immunoassay (CLIA )	pmol/L	11.3	9.1-13.5	
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA )	pmol/L	15	12-18	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	pmol/L	11.3	8.9-13.7	
Caris200 WanTai \ Chemiluminescence Immunoassay (CLIA )	pmol/L	8.36	6.35-10.4	
<b>(FT4 )游离T4</b>				
Tosoh AIA1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	pmol/L	34.2	27-41.4	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	pmol/L	31.5	25.2-37.8	
Siemens ADVIA Centaur XP Siemens \ Chemiluminescence enzyme immunoassay (CLEIA)	pmol/L	32.2	25.8-38.6	
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA )	pmol/L	26.5	21.4-31.6	

<b>REF</b> 540-2	6x3mL	<b>EXP</b> 2025-02-15	<b>LOT</b> 3384FD6	更新日期： 2022-05 4 / 6
------------------	-------	-----------------------	--------------------	------------------------

批号：3384FD6				
项目\仪器\试剂\方法	单位	均值	+ / - 2 SD	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	pmol/L	29.5	23.7-35.3	
Caris200 WanTai \ Chemiluminescence Immunoassay (CLIA )	pmol/L	25	19-31	
<b>(hCG)人绒毛膜促性腺激素</b>				
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	IU/L	147	119-175	
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA )	IU/L	152	124-180	
<b>(HCY)同型半胱氨酸</b>				
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA )	μmol/L	24.3	19.7-28.9	
Roche cobas 4000/6000/8000 Roche \ Enzymatic Cycling Assays Method	μmol/L	22.8	18.2-27.4	
Tosoh AIA 1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	μmol/L	22.1	17.3-26.9	
<b>(HGH)人生长激素</b>				
Beckman UniCel Dxl 600/800 Dedicated \ Chemiluminescence Immunoassay (CLIA )	μg/L	3.53	2.85-4.21	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	μg/L	3.53	2.85-4.21	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	μg/L	4.2	3.36-5.04	
Tosoh AIA 1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	μg/L	3.97	3.17-4.77	
<b>(IgE)免疫球蛋白 IgE</b>				
Beckman Image 800 Beckman Coulter \ Immunoturbidimetry (ITA)	IU/mL	79.1	63.3-94.9	
Tosoh AIA 1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	IU/mL	80.6	64.2-97	
<b>(IGF1)胰岛素生长因子1</b>				
Autolumo A2000/A2000 PLUS Autobio \ Chemiluminescence Immunoassay (CLIA )	ng/mL	211	169-253	
<b>(INS)胰岛素</b>				
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA )	mIU/L	25.1	20.2-30	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	mIU/L	34.7	27.5-41.9	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	mIU/L	33.3	26.6-40	
Siemens ADVIA Centaur XP Siemens \ Chemiluminescence Immunoassay (CLIA )	mIU/L	31.2	24.8-37.6	
Tosoh AIA 1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	mIU/L	32.1	25.3-38.9	
<b>(LH)促黄体生成素</b>				
Caris200 WanTai \ Chemiluminescence Immunoassay (CLIA )	IU/L	32.1	24.4-39.8	
Tosoh AIA1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	IU/L	17.9	14.1-21.7	
Siemens Immulite1000/2000 Siemens \ Chemiluminescence Immunoassay (CLIA )	IU/L	28.5	22.9-34.1	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/L	22.9	18.3-27.5	
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA )	IU/L	19.3	15.5-23.1	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	IU/L	16.6	12.4-20.8	
<b>(Myo / Mb)肌红蛋白</b>				
Tosoh AIA1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	μg/L	143	115-171	
Roche cobas e411 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	μg/L	152	122-182	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	μg/L	126	102-150	

<b>REF</b> 540-2	6x3mL	<b>EXP</b> 2025-02-15	<b>LOT</b> 3384FD6	更新日期： 2022-05 5 / 6
------------------	-------	-----------------------	--------------------	------------------------

批号：3384FD6				
项目\仪器\试剂\方法	单位	均值	+ / - 2 SD	
<b>(NT-proBNP)N-末端脑钠肽前体</b>				
Roche cobas e411 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	pg/mL	1270	1010-1520	
Roche cobas e411 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	pmol/L	150	120-180	
<b>(PRL)催/泌乳素</b>				
Siemens Immulite1000/2000 Siemens \ Chemiluminescence Immunoassay (CLIA)	ng/mL	15.9	13.1-18.7	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	18.6	14.9-22.3	
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA)	ng/mL	15.2	12.2-18.2	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	14.1	11.3-16.9	
Caris200 WanTai \ Chemiluminescence Immunoassay (CLIA)	ng/mL	16	12.2-19.8	
<b>(PROG)孕酮</b>				
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	10.6	8.34-12.9	
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA)	ng/mL	11.2	9-13.4	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	11.6	9.29-13.9	
Siemens Immulite1000/2000 Siemens Dade Actin FS \ Chemiluminescence enzyme immunoassay (CLEIA)	ng/mL	8.23	6.59-9.87	
Tosoh AIA 1800 TOSOH \ Chemiluminescence Immunoassay (CLIA)	ng/mL	11.6	9.15-14.1	
<b>(PSA)前列腺特异性抗原</b>				
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA)	ng/mL	12.7	10.3-15.1	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	12.5	9.7-15.3	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	14	11.2-16.8	
Tosoh AIA1800 TOSOH \ Chemiluminescence Immunoassay (CLIA)	ng/mL	11.5	8.9-14.1	
Caris200 WanTai \ Chemiluminescence Immunoassay (CLIA)	ng/mL	15.9	12.1-19.7	
<b>(PTH)甲状旁腺素</b>				
Tosoh AIA1800 TOSOH \ Chemiluminescence Immunoassay (CLIA)	pg/mL	90.2	0-269	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	pg/mL	86.7	69.4-104	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	pg/mL	80.4	63.8-97	
<b>(SHBG)性激素结合球蛋白</b>				
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	nmol/L	78.8	59.9-97.7	
<b>(TES)睾酮</b>				
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	0.82	0.64-1	
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA)	ng/mL	0.42	0.34-0.5	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	0.474	0.38-0.568	
Siemens Immulite1000/2000 Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	ng/mL	0.59	0.47-0.71	
<b>(Tg)甲状腺球蛋白</b>				
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA)	ng/mL	7.3	5.5-9.1	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	11.1	8.88-13.3	
Siemens ADVIA Centaur XP Siemens \ Chemiluminescence enzyme immunoassay (CLEIA)	ng/mL	2.37	1.91-2.83	

<b>REF</b> 540-2	6x3mL	<b>EXP</b> 2025-02-15	<b>LOT</b> 3384FD6	更新日期： 2022-05 6 / 6
------------------	-------	-----------------------	--------------------	------------------------

批号：3384FD6				
项目\仪器\试剂\方法	单位	均值	+ / - 2 SD	
<b>(TnI)肌钙蛋白I</b>				
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	ng/mL	0.42	0.3-0.54	
Tosoh AIA 1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	ng/mL	0.47	0.35-0.59	
<b>(TnT)肌钙蛋白T</b>				
Roche cobas e411 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	0.914	0.731-1.1	
<b>(TSH)促甲状腺素</b>				
Beckman UniCel Dxl 600/800 Beckman Coulter \ Electro-Chemiluminescence Immunoassay (ECLIA)	mIU/L	10.7	8.3-13.1	
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA )	mIU/L	8.76	6.96-10.6	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	mIU/L	11.7	9.35-14	
Tosoh AIA1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	mIU/L	9.78	8.06-11.5	
Caris200 WanTai \ Chemiluminescence Immunoassay (CLIA )	mIU/L	10	7.6-12.4	
<b>(TT3)总三碘甲状原氨酸</b>				
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	ng/mL	2.02	0-6.07	
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA )	ng/mL	2.15	1.71-2.59	
Siemens ADVIA Centaur XP Siemens \ Chemiluminescence Immunoassay (CLIA )	ng/mL	2.82	2.26-3.38	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	2.43	1.94-2.92	
Caris200 WanTai \ Chemiluminescence Immunoassay (CLIA )	ng/mL	1.64	1.25-2.03	
<b>(TT4)总T4</b>				
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	nmol/L	153	127-179	
Abbott ARCHITECT i series Abbott \ Chemiluminescence Immunoassay (CLIA )	nmol/L	153	125-181	
Siemens ADVIA Centaur XP Siemens \ Chemiluminescence Immunoassay (CLIA )	nmol/L	175	142-208	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	nmol/L	156	125-187	
Caris200 WanTai \ Chemiluminescence Immunoassay (CLIA )	nmol/L	132	100-164	
<b>(VB12)维生素B12</b>				
Tosoh AIA 1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	pg/mL	659	524-794	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	pg/mL	718	574-862	
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	pg/mL	567	455-679	
<b>(β2M)β-2-微球蛋白</b>				
Roche cobas 4000/6000/8000 Roche \ Immunoturbidimetry (ITA)	mg/L	1.36	1.09-1.63	
Tosoh AIA1800 TOSOH \ Chemiluminescence Immunoassay (CLIA )	mg/L	1.44	1.12-1.76	
<b>(β-hCG)人绒毛膜促性腺激素β亚基</b>				
Beckman UniCel Dxl 600/800 Beckman Coulter \ Chemiluminescence Immunoassay (CLIA )	IU/L	139	111-167	
Roche cobas E601/602 Roche \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/L	154	123-185	
Caris200 WanTai \ Chemiluminescence Immunoassay (CLIA )	IU/L	166	126-206	

注：此报告所提供的数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来，仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的改变，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。