

<b>REF</b> 539-1	6x3mL	<b>EXP</b> 2024-08-04	<b>LOT</b> 336A22D	更新日期： 2022-11 1 / 5
------------------	-------	-----------------------	--------------------	------------------------

本报告含以下项目的示范实验室数据

参数 ( Parameters )	参数 ( Parameters )	参数 ( Parameters )
(17-OHP)17-羟孕酮	(AFP)甲胎蛋白	(ALD)醛固酮
(Anti-Tg)抗甲状腺球蛋白抗体	(Anti-TPO)抗甲状腺过氧化物酶抗体	(CA 125)糖类抗原 125
(CA 15-3)糖类抗原 15-3	(CA 19-9)糖类抗原 19-9	(CEA)癌胚抗原
(Cortisol)皮质醇	(Cyfra 21-1)细胞角蛋白19片段	(DHEA-S)硫酸脱氢表雄酮
(E2)雌二醇	(FA)叶酸	(Ferritin)铁蛋白
(F-PSA)游离前列腺特异性抗原	(FSH)促卵泡刺激素	(FT3)游离T3
(FT4 )游离T4	(hCG)人绒毛膜促性腺激素	(HCY)同型半胱氨酸
(IgE)免疫球蛋白 IgE	(IGF1)胰岛素生长因子1	(INS)胰岛素
(LH)促黄体生成素	(PRL)催乳素	(PROG)孕酮
(PSA)前列腺特异性抗原	(PTH)甲状旁腺素	(TES)睾酮
(Tg)甲状腺球蛋白	(THEO)茶碱	(TSH)促甲状腺素
(TT3)总三碘甲状腺氨酸	(TT4)总T4	(β2M)β-2-微球蛋白
(β-hCG)人绒毛膜促性腺激素β亚基		

注：此报告所提供的项目与数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来。此报告不可代替产品说明书。此报告仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的改变，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。

<b>REF</b> 539-1	6x3mL	<b>EXP</b> 2024-08-04	<b>LOT</b> 336A22D	更新日期： 2022-11 2 / 5
------------------	-------	-----------------------	--------------------	------------------------

批号：336A22D				
项目\仪器\试剂方法	单位	均值	+ / - 2 SD	
<b>(17-OHP)17-羟孕酮</b>				
Siemens BEP III Dedicated \ Enzyme linked immunosorbent assays (ELISA)	ng/mL	1.59	1.33-1.85	
<b>(AFP)甲胎蛋白</b>				
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	7.95	6.46-9.44	
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	7.21	5.77-8.65	
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	5.39	4.33-6.45	
Siemens ADVIA Centaur XP Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	ng/mL	8.51	6.65-10.4	
<b>(ALD)醛固酮</b>				
Siemens BEP III Dedicated \ Enzyme immunoassay (EIA)	pg/mL	56.6	0-169	
<b>(Anti-Tg)抗甲状腺球蛋白抗体</b>				
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/mL	5.89	4.73-7.05	
Beckman Access Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/mL	2.06	1.64-2.48	
Beckman UniCel Dxl 600/800 Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/mL	1.89	1.47-2.31	
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/mL	26.2	21-31.4	
Siemens ADVIA Centaur XP Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	IU/mL	48.1	38.5-57.7	
<b>(Anti-TPO)抗甲状腺过氧化物酶抗体</b>				
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/mL	10.1	7.9-12.3	
Beckman Access Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/mL	7.08	5.66-8.5	
Beckman UniCel Dxl 600/800 Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/mL	6.56	5.32-7.8	
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/mL	7.65	6.19-9.11	
Siemens ADVIA Centaur XP Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	IU/mL	74.2	59.4-89	
<b>(CA 125)糖类抗原 125</b>				
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/mL	47.7	38.5-56.9	
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/mL	37.3	30.9-43.7	
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/mL	35.2	27.8-42.6	
Siemens ADVIA Centaur XP Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	IU/mL	33.5	27.1-39.9	
Siemens Immulite1000/2000 Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	IU/mL	33.1	26.7-39.5	
<b>(CA 15-3)糖类抗原 15-3</b>				
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/mL	23.2	18-28.4	
Beckman Access Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/mL	16.4	11.2-21.6	
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/mL	21.9	17.7-26.1	
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/mL	22.7	17.7-27.7	
Siemens ADVIA Centaur XP Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	IU/mL	26.3	21.1-31.5	

注：此报告所提供的项目与数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来。此报告不可代替产品说明书。此报告仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的改变，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。

<b>REF</b> 539-1	6x3mL	<b>EXP</b> 2024-08-04	<b>LOT</b> 336A22D	更新日期： 2022-11 3 / 5
------------------	-------	-----------------------	--------------------	------------------------

(CA 19-9)糖类抗原 19-9				
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/mL	25.6		20.2-31
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/mL	21.5		17.3-25.7
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/mL	22.3		17.7-26.9
Siemens ADVIA Centaur XP Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	IU/mL	19.6		15.8-23.4
Siemens Immulite1000/2000 Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	IU/mL	34.2		27.4-41
(CEA)癌胚抗原				
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	4.79		3.91-5.67
Beckman Access Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	5.53		4.41-6.65
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	4.75		3.83-5.67
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	4.68		3.76-5.6
Siemens Immulite1000/2000 Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	ng/mL	6.25		4.99-7.51
(Cortisol)皮质醇				
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	111		89-133
(Cyfra 21-1)细胞角蛋白19片段				
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	1.21		1.03-1.39
(DHEA-S)硫酸脱氢表雄酮				
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	2200		1880-2510
(E2)雌二醇				
Beckman Access Dedicated \ Chemiluminescence Immunoassay (CLIA )	pmol/L	209		205-213
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	pmol/L	194		162-226
Siemens Immulite1000/2000 Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	pmol/L	142		116-168
(FA)叶酸				
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	2.22		1.69-2.75
Beckman Access Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	4.26		3.46-5.06
(Ferritin)铁蛋白				
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	µg/L	47.2		37.4-57
Beckman Access Dedicated \ Chemiluminescence Immunoassay (CLIA )	µg/L	34		27.1-40.9
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	µg/L	47.2		37.4-57
Siemens ADVIA Centaur XP Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	µg/L	42.5		41.6-43.4
(F-PSA)游离前列腺特异性抗原				
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	1.16		0.94-1.38
(FSH)促卵泡刺激素				
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/L	5.64		4.56-6.72
(FT3)游离T3				
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	pmol/L	4.23		3.37-5.09

注：此报告所提供的项目与数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来。此报告不可代替产品说明书。此报告仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的改变，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。

<b>REF</b> 539-1	6x3mL	<b>EXP</b> 2024-08-04	<b>LOT</b> 336A22D	更新日期： 2022-11 4 / 5
------------------	-------	-----------------------	--------------------	------------------------

Beckman Access Dedicated \ Chemiluminescence Immunoassay (CLIA )	pmol/L	4.15	3.31-4.99
Beckman UniCel Dxl 600/800 Dedicated \ Chemiluminescence Immunoassay (CLIA )	pmol/L	4.11	3.29-4.93
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	pmol/L	5.14	3.91-6.37
<b>(FT4 )游离T4</b>			
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	pmol/L	14.2	11.4-17
Beckman UniCel Dxl 600/800 Dedicated \ Chemiluminescence Immunoassay (CLIA )	pmol/L	12.8	10.6-15
Roche Modular E170 Dedicated \ Chemiluminescence Immunoassay (CLIA )	pmol/L	16	12.7-19.3
<b>(hCG)人绒毛膜促性腺激素</b>			
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/L	5.45	4.33-6.57
Siemens ADVIA Centaur XP Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/L	4.34	3.46-5.22
Tosoh AIA1800 Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/L	5.81	4.83-6.79
<b>(HCY)同型半胱氨酸</b>			
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	μmol/L	14.6	11.8-17.4
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	μmol/L	13.7	11.3-16.1
<b>(IgE)免疫球蛋白 IgE</b>			
Beckman Immage 800 Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/mL	33.8	27-40.6
Beckman Immage 800 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/mL	55.8	45-66.6
<b>(IGF1)胰岛素生长因子1</b>			
Siemens Immulite1000/2000 Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	ng/mL	152	124-180
<b>(INS)胰岛素</b>			
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	mIU/L	12.6	10.4-14.8
<b>(LH)促黄体生成素</b>			
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/L	3.44	2.84-4.04
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/L	4.93	4.07-5.79
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	IU/L	4.25	3.39-5.11
Siemens Immulite1000/2000 Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	IU/L	4.34	3.46-5.22
<b>(PRL)催/ 泌乳素</b>			
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	8.4	6.8-10
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	8.67	7.11-10.2
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	7.68	6.28-9.08
<b>(PROG)孕酮</b>			
Beckman Access Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	0.836	0.672-1
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	0.843	0.667-1.02
Siemens ADVIA Centaur XP Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	ng/mL	0.77	0.442-1.1
<b>(PSA)前列腺特异性抗原</b>			
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	3.28	2.5-4.06

注：此报告所提供的项目与数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来。此报告不可代替产品说明书。此报告仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的改变，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。

<b>REF</b> 539-1	6x3mL	<b>EXP</b> 2024-08-04	<b>LOT</b> 336A22D	更新日期： 2022-11 5 / 5
------------------	-------	-----------------------	--------------------	------------------------

Beckman Access Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	4.19	3.35-5.03
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	3.96	3.3-4.62
Roche Modular E170 Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	4.21	3.41-5.01
<b>(PTH)甲状旁腺素</b>			
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	pg/mL	18.3	14.9-21.7
<b>(TES)睾酮</b>			
Abbott ARCHITECT i series Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	7.41	6.03-8.79
Beckman Access Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	6.5	5.3-7.7
Roche cobas 4000/6000/8000 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	6.47	5.26-7.68
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	6.32	5.11-7.53
<b>(Tg)甲状腺球蛋白</b>			
Beckman UniCel Dxl 600/800 Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	13.3	10.7-15.9
Roche cobas e411 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	ng/mL	22.3	17.9-26.7
<b>(THEO)茶碱</b>			
Siemens ADVIA Centaur XP Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	µg/mL	10.8	8.6-13
<b>(TSH)促甲状腺素</b>			
Abbott ARCHITECT i series Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	mIU/L	13.5	10.7-16.3
Roche Modular E170 Dedicated \ Electro-Chemiluminescence Immunoassay (ECLIA)	mIU/L	15.2	12-18.4
<b>(TT3)总三碘甲状腺原氨酸</b>			
Beckman Access Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	1.09	0.856-1.32
Beckman UniCel Dxl 600/800 Dedicated \ Chemiluminescence Immunoassay (CLIA )	ng/mL	1.03	0.86-1.2
Siemens ADVIA Centaur XP Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	ng/mL	1.21	0.85-1.57
<b>(TT4)总T4</b>			
Beckman Access Dedicated \ Chemiluminescence Immunoassay (CLIA )	nmol/L	81.5	65.1-97.9
Beckman UniCel Dxl 600/800 Dedicated \ Chemiluminescence Immunoassay (CLIA )	nmol/L	72.2	59.4-85
<b>(β2M)β-2-微球蛋白</b>			
Roche cobas 4000/6000/8000 Dedicated \ Immunonephelometry (INA)	mg/L	12.7	10.3-15.1
<b>(β-hCG)人绒毛膜促性腺激素β亚基</b>			
Beckman Access Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/L	4.21	3.41-5.01
Roche Modular E170 Dedicated \ Chemiluminescence enzyme immunoassay (CLEIA)	IU/L	3.56	2.8-4.32
Tosoh AIA 1800 Dedicated \ Chemiluminescence Immunoassay (CLIA )	IU/L	5.78	4.98-6.58

注：此报告所提供的项目与数据均基于检测相同批号质控品的若干实验室的数据汇总统计而来。此报告不可代替产品说明书。此报告仅供学习、参考之用。因所用技术、仪器和试剂的不同，或因制造商检测方法的改变，均可导致实验室实际测得的数据偏离此报告所提供的数据。根据良好实验室规范的要求，实验室须遵循相关技术规范确立自己的均值和可接受范围。